



ANBOINI SYSTEM

Smart Lingual Orthodontic Solutions

**Anboini**TM
Lingual Bracket

Multi-slotted Lingual Bracket and Lingual Straight Wire

ANBOINI IS THE ONLY SYSTEM IN THE WORLD THAT COMBINES

Completely Customized Multi-slotted Lingual Bracket

Comprehensive Line of Preformed Lingual Straight Archwires

Individual Transfer Tray

Helps you achieve board-standard finishes with multi-slotted lingual bracket


Makes treatment easier with preformed lingual straight wire



Multi-Slotted Lingual Bracket

Anboini is a multi-slotted lingual bracket system that offers exceptional control in all three dimensions, which helps us achieve board-standard finishes.


For best result, effective control of tooth movements such as angulation, rotation, and torque is important in orthodontics. Therefore, Anboini bracket has three different types of slot: horizontal slot for main archwire, vertical slot for rotation control, and accessory slot for angulation control. In addition, tandem archwire technique can be used for anterior torque control during retraction.

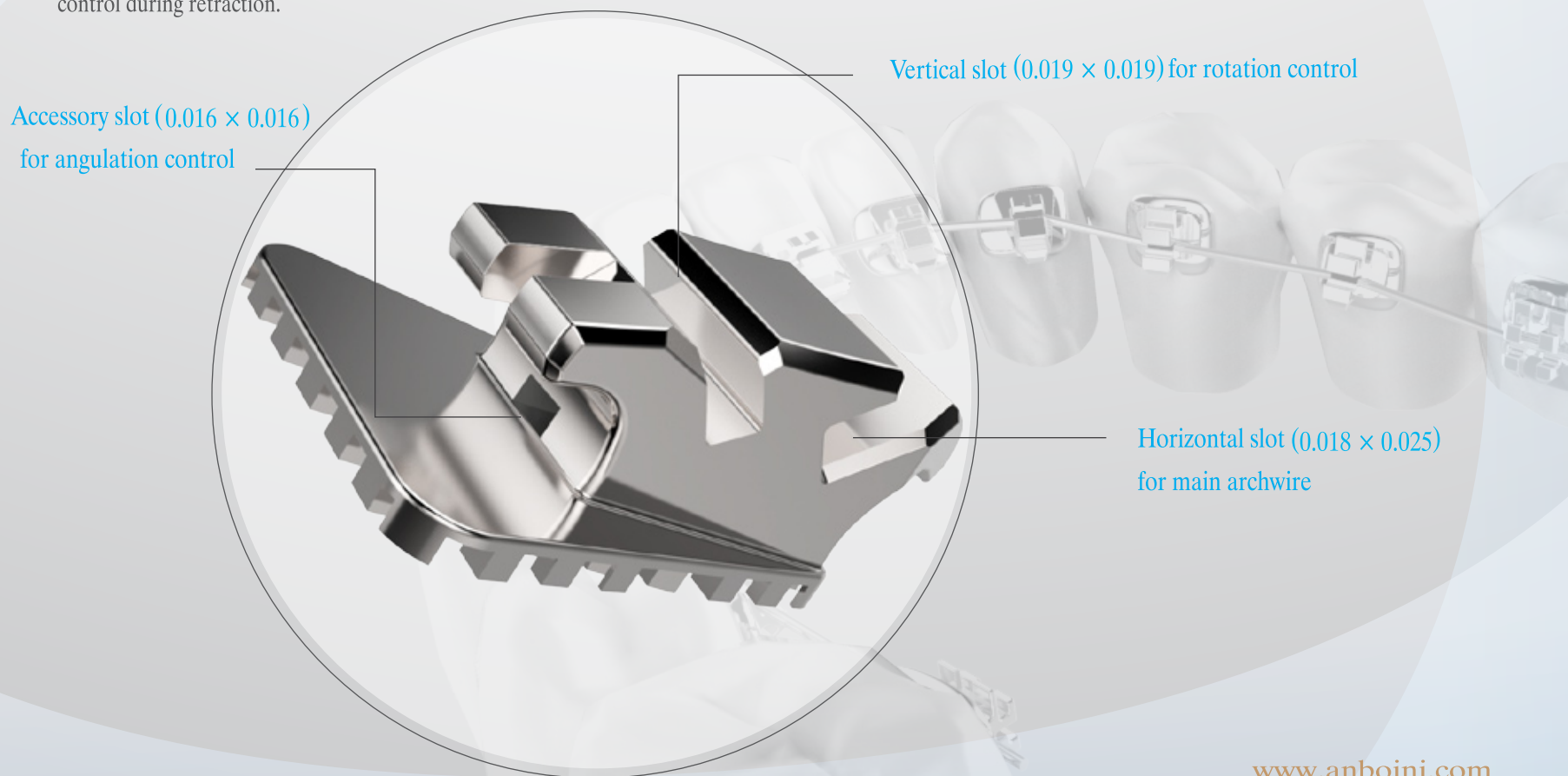

Preformed Lingual Straight Archwire

Anboini is the only lingual system that provides a series of preformed straight archwires which help us make the treatment more convenient.


Using upper 147 and lower 71 individual clinical lingual archwires from ideal setup models, 3 groups of the lingual straight archwire form were clustered. Therefore, 3 types (small, medium, and large) of the lingual straight archwire form are prepared, and one of them is used for maxilla and/or mandible.

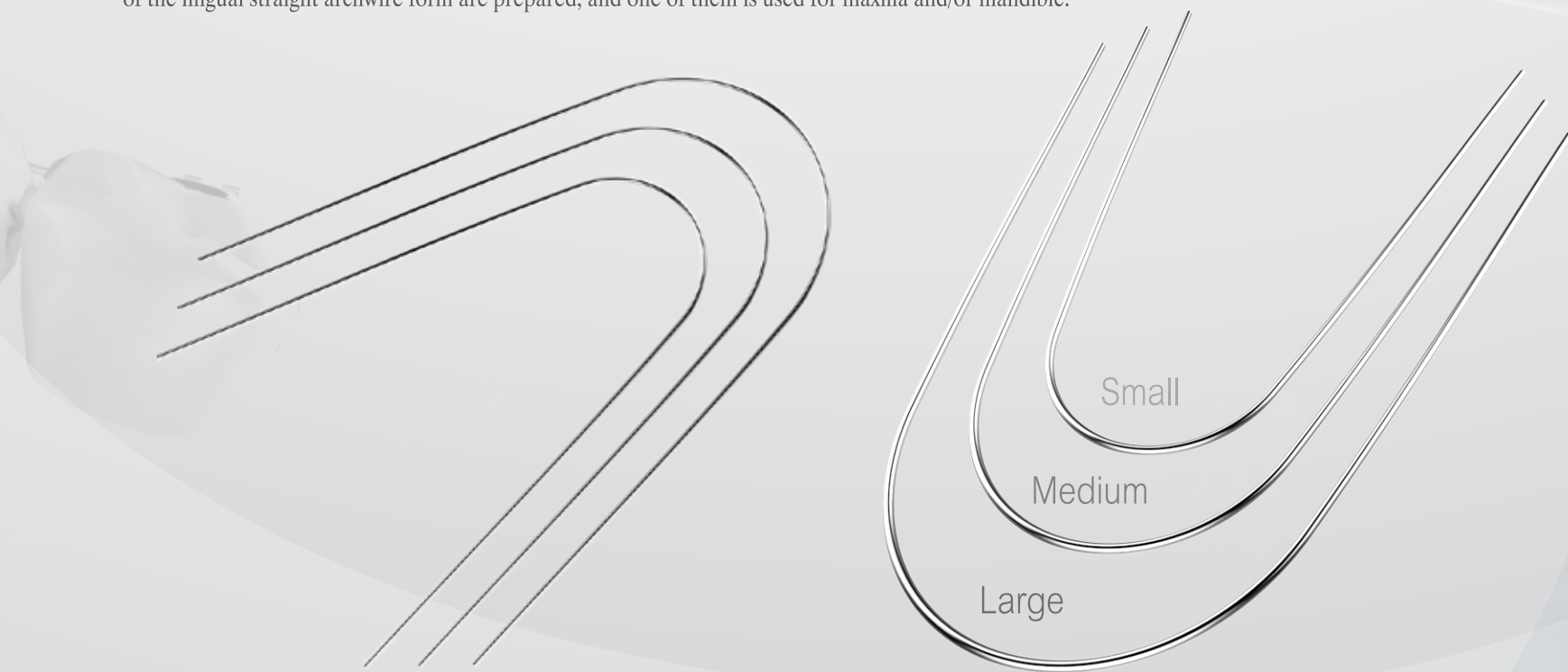

Individual Transfer Tray

The Anboini transfer tray¹ is individual, which consists of bracket index resin (BIR) and tooth index resin (TIR) although group transfer tray for groups of adjacent teeth is used for indirect bonding in other competitor systems. Because individual transfer tray is made directly from the setup model, the brackets are precisely and efficiently delivered to the patient's mouth.

1. Hong RK, Kim YH, and Park JY. A new customized lingual indirect bonding system. J Clin Orthod 2000;34:456-60.

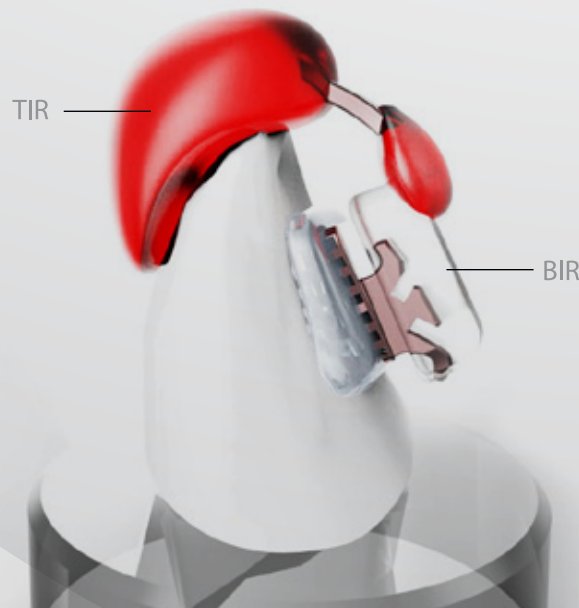

The advantages of using individual transfer trays over group transfer trays include

Accuracy - Precise indirect bonding

Reproducibility - Easy and precise Rebonding

Visibility - Easy verification of goodness of fit

Accessibility - Easy access for curing light and removing excess bonding resin


Process A to Z

The following outlines how to start a case with the Anboini system.

We can offer well-priced Anboini system (customized lingual bracket, preformed straight archwire, and individual transfer tray), which will be arrived 3~4 weeks after we receive your order.


Please visit www.anboini.com for submitting your case.

“You can enjoy the lingual orthodontic treatment with Anboini system.”

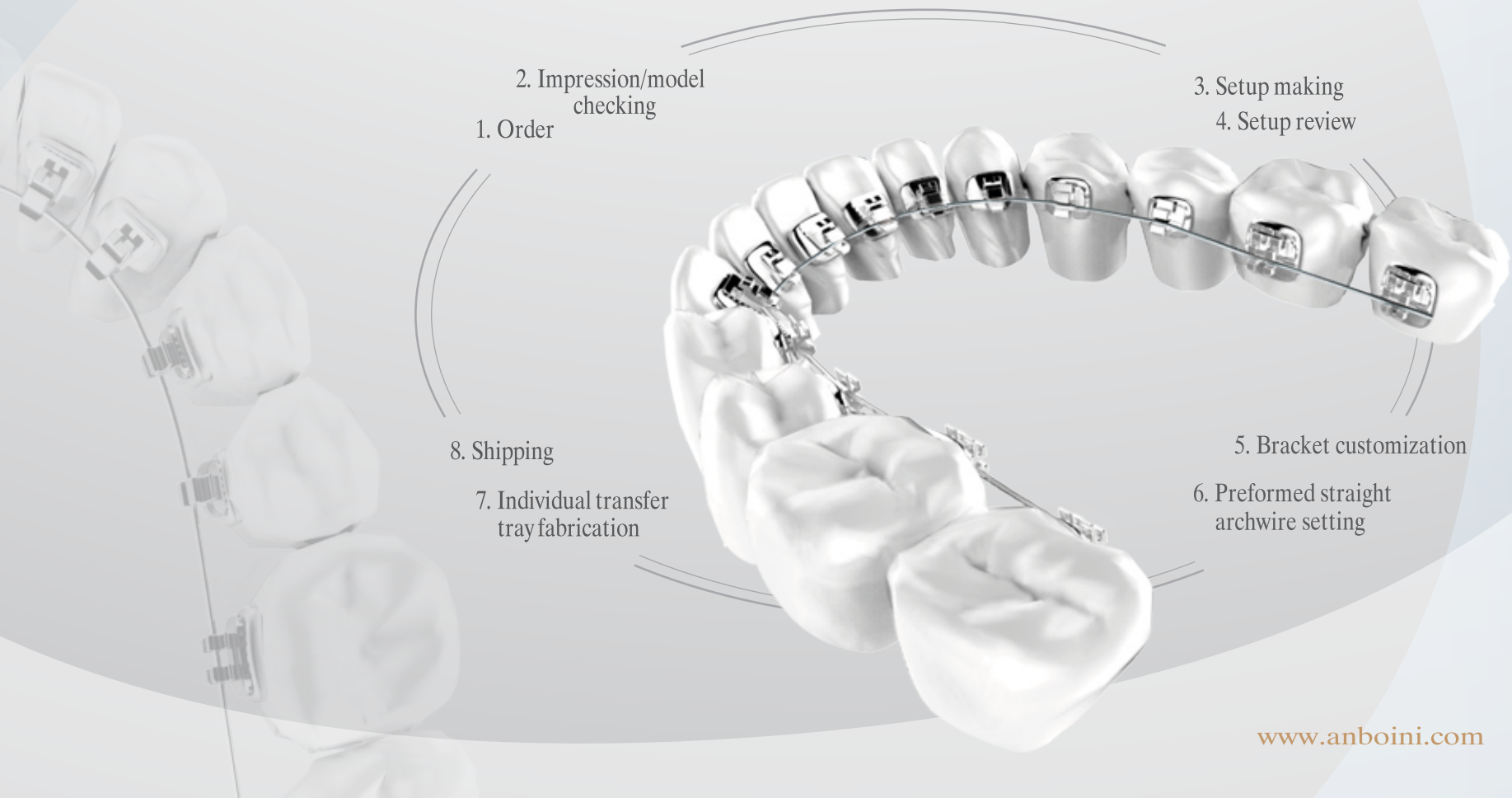

CASE

Extraction Case

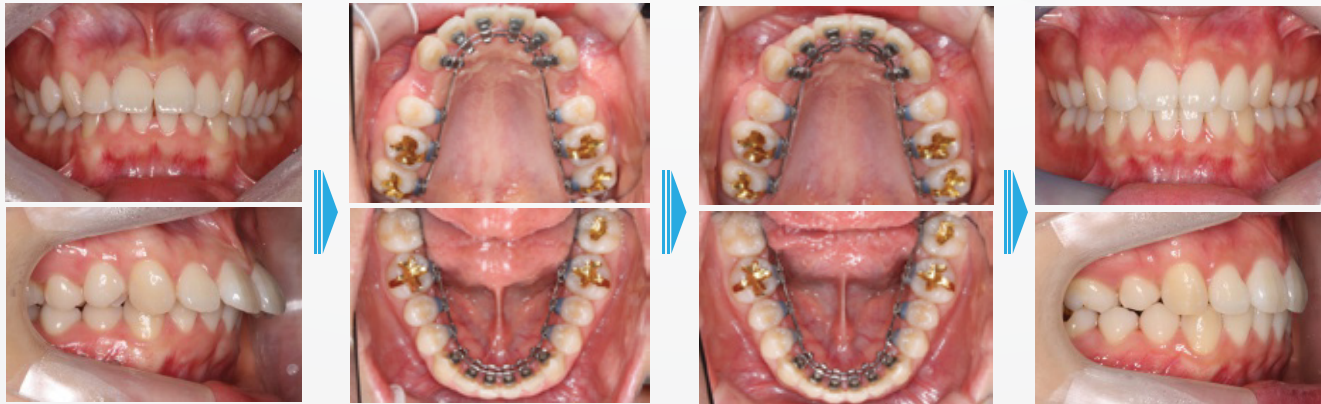


BEFORE TREATMENT



AFTER TREATMENT

Non-extraction Case

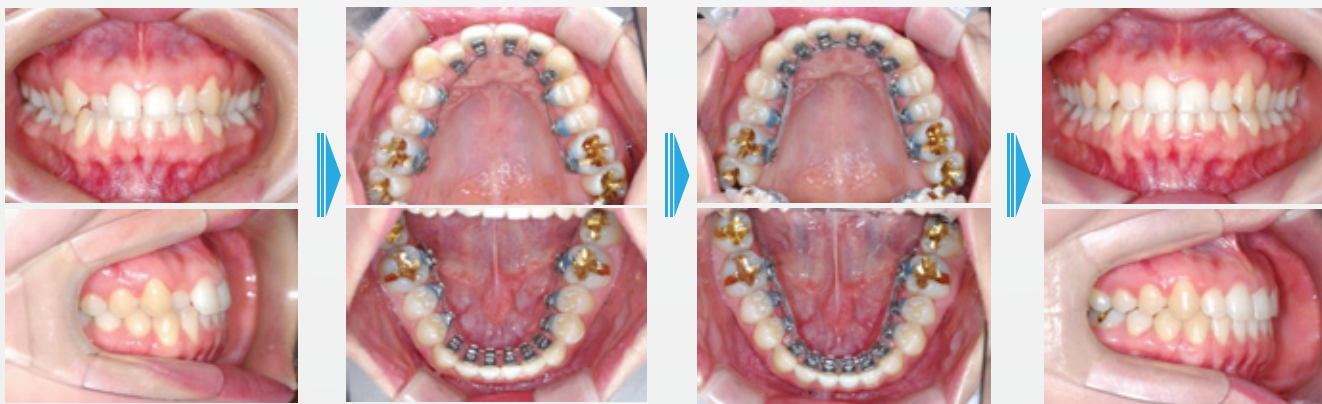

BEFORE TREATMENT



AFTER TREATMENT

AnboiniTM
Lingual Bracket
www.anboini.com



Head Office

Room. 512, Kolon Science ValleyII, 55, Digital-ro 34-gil, Guro-gu, seoul,
152-728, Korea

TEL : +82-2-2025-0721,5 [domestic] +82-2-2025-0726 [international]

FAX : 82-2-2025-0728

International Marketing & Sales Dept. **Andy OH**

TEL : +82-2-2025-0726 E-mail : andyny@biomk.com / andybma@gmail.com

Web site. www.biomk.com Facebook. www.facebook.com/biomaterialskorea

Online store. www.biomkstore.com